

# Microvette®

Diagnostic Products

## *Capillary blood collection systems – As individual as your patients*

### **Microvette® 200 – Capillary blood collection**

The new Microvette® 200 capillary blood collection system is designed to offer practical solutions for individual requirements. Depending on your requirements, you can choose between the Microvette® 200 with a conical or round bottom inner tube. For both versions the end-to-end capillary is preassembled and ready for use.

The Microvette® 200 offers all the advantages of a modern capillary blood collection system:

- Assembled capillary for blood collection according to the End-to-End principle
- Special twist cap design is easy to open and minimizes aerosol effect
- Optimized tube geometry ensures thorough sample mixing
- Colour-coded caps and print for easy identification of additives and volume



Blood collection with assembled 200 µl end-to-end capillary

### **Microvette® 300/500 – Blood collection with collection rim**

The new Microvette® tubes are ideal for the collection of blood using the gravity-flow principle of collection. Any part of the entire rim can be used. The special conical design of the Microvette® 300's inner tube enables thorough mixing, even with small sample volumes.

The Microvette® offers all the advantages of a modern blood collection system:

- Choice of different inner tube designs with 300 or 500 µl volume capacity and corresponding graduation
- Special twist cap design is easy to open and minimizes aerosol effect
- Cylindrical outer container is ideal for barcode or patient labels



Simple peripheral blood collection thanks to newly developed collection vessel



Assembled End-to-End capillary and graduation

### Microvette® 200

200 µl sample volume, conical inner tube

| Preparation                | Order Number |
|----------------------------|--------------|
| Serum / Clotting activator | 20.1290      |
| Plasma / Lithium Heparin   | 20.1292      |
| Glucose / Fluoride         | 20.1289      |

### Microvette® 200

200 µl sample volume, round bottom inner tube

| Preparation                    | Order Number |
|--------------------------------|--------------|
| Serum gel / Clotting activator | 20.1291      |
| Haematology / Potassium EDTA   | 20.1288      |

### Microvette® CB 200

for ESR

| Preparation              | Order Number |
|--------------------------|--------------|
| Citrate                  | 18.1325      |
| ESR rack with graduation | 90.1091      |



Sample volume 300 µl or 500 µl

### Microvette® 300

300 µl sample volume, conical inner tube

| Preparation                | Order Number |
|----------------------------|--------------|
| Serum / Clotting activator | 20.1308      |
| Plasma / Lithium Heparin   | 20.1309      |
| Glucose / Fluoride         | 20.1307      |

### Microvette® 500

500 µl sample volume, round bottom inner tube

| Preparation                    | Order Number |
|--------------------------------|--------------|
| Serum / Clotting activator     | 20.1343      |
| Serum gel / Clotting activator | 20.1344      |
| Plasma / Lithium Heparin       | 20.1345      |
| Plasma / Lithium Heparin Gel   | 20.1346      |
| Haematology / Potassium EDTA   | 20.1341      |

Your Distributor is:  
Braintree Scientific, Inc.  
PO Box 850498, Braintree, MA 02185  
781-917-9526  
Email: [Info@braintreesci.com](mailto:Info@braintreesci.com)  
Web: [www.braintreesci.com](http://www.braintreesci.com)

# Multivette® 600, Microvette® CB 300

## Capillary and venous blood collection in one system


### Multivette® 600

For venous blood collection, a Luer needle is connected to the capillary tube of the Multivette®. The special design enables self-filling of the inner tube by normal venous pressure.


Hygienic and safe capillary blood collection is performed using the End-to-End principle. The special capillary tube is designed to hold a volume of 600 µl of capillary blood.

- Venous blood collection is performed by connection of a luer needle\* to the capillary tube of the Multivette®
- Hygienic and safe capillary blood collection according to the end-to-end principle
- Graduations ensure volume control
- Maximum volume 600 µl

\*When collecting blood with a Luer needle, make sure to remove the needle from the vein together with the assembled Multivette® after blood collection.

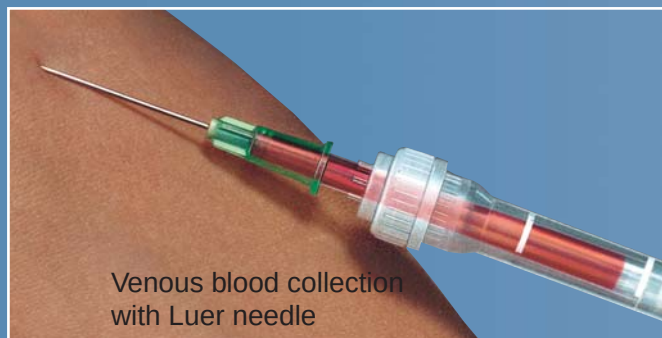

Venous blood collection with Luer needle

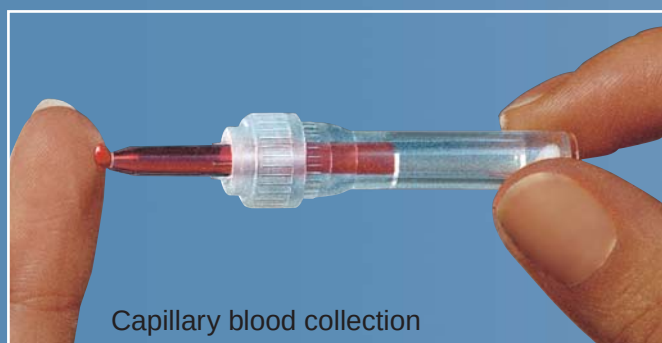

Capillary blood collection

## The system for optimum serum/plasma recovery

### Microvette® CB 300


The capillary tube with push cap is supplied complete with a carrier tube that is also suited for transport and centrifugation.

After centrifugation, the capillary tube is sealed with the cap attached to the push cap.

The trumpet-shaped capillary tube ensures optimum serum/plasma recovery after centrifugation.

For efficient working in the laboratory, the capillary tube accommodated in its carrier tube can be placed directly into the analyser.

- Optimum serum/plasma recovery after centrifugation
- Two in one - sample container and centrifuge tube
- Compatible with different analysers as a primary tube for direct sampling

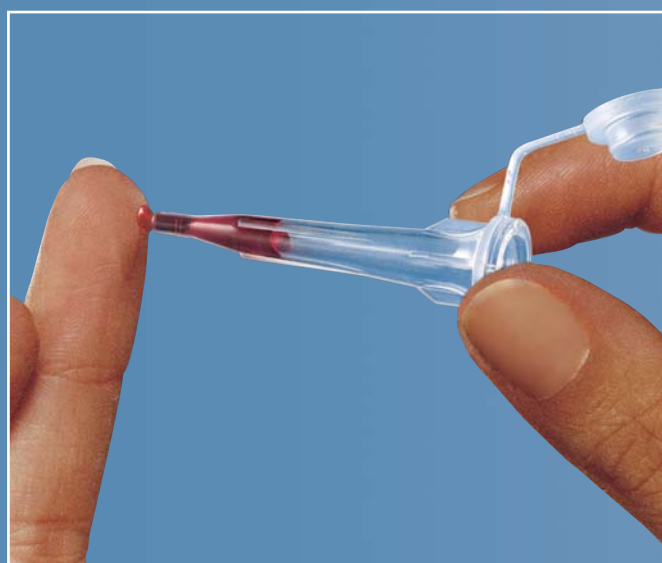

Easy sampling  
Volumes up to 300 µl



Due to the colour-coded caps of the capillary tube and the attached screw-cap, the risk of sample mix-ups is eliminated.

## Multivette® 600

600 µl sample volume

| Preparation                    | Order Number |
|--------------------------------|--------------|
| Serum / Clotting activator     | 15.1670      |
| Serum gel / Clotting activator | 15.1674      |
| Plasma / Lithium Heparin       | 15.1673      |
| Plasma gel / Lithium Heparin   | 15.1675      |
| Haematology / Potassium EDTA   | 15.1671      |
| Glucose / Fluoride             | 15.1672      |

Multivettes are packed in units of 100 (1,000/case)

## Carrier tube for Multivette® 600

- Safe sample identification by application of patient-related or barcode labels
- Suitable for centrifugation
- Also available in brown (light protection)

|                    |               |             |
|--------------------|---------------|-------------|
| <b>transparent</b> | Order Number: | 55.1577     |
| <b>dyed brown</b>  | Order Number: | 55.1577.002 |



## Microvette® CB 300

| Preparation                                  | Order Number |
|----------------------------------------------|--------------|
| Serum / Clotting activator                   | 16.440       |
| Serum / Clotting activator <b>dyed brown</b> | 16.441*      |
| Plasma / Lithium Heparin                     | 16.443       |
| Plasma / Lithium Heparin <b>dyed brown</b>   | 16.434*      |
| Haematology / Potassium EDTA                 | 16.444       |
| Glucose / Fluoride                           | 16.446       |

Pigmented brown for bilirubin determination

Microvettes are packaged in units of 50 (1,000/case).

- Easy drawing off of the supernatant
- Optimum serum/plasma recovery after centrifugation
- Capillary tube supplied with carrier tube for sampling transport and centrifugation in one system

Your Distributor is:  
Braintree Scientific, Inc.  
PO Box 850498, Braintree, MA 02185  
781-917-9526  
Email: [Info@braintreesci.com](mailto:Info@braintreesci.com)  
Web: [www.braintreesci.com](http://www.braintreesci.com)