

Gaymar Connector Types Explained

Clik-Tite Connector



Colder Style Connector



Gaymar Click-Tite and Push-Button Colder connectors

The TP-700 pump includes Clik-Tite connectors which are compatible with all green Gaymar pads. The T-Pump Reusable pads feature push-button Colder Style connectors which are not compatible with Clik-Tites.

You can purchase the connector type you need for any pad. Simply cut off the original connector, dip the hose end in warm water to soften it, and then push the barb end of the new connector into the hose.

Clik-Tite Models:	TP3E, TP12E, TP22E, TP26E, TP22B, TP22C, TP22G
--------------------------	---

Colder (Push-Button) Models:	121218, 122922, 125923
-------------------------------------	------------------------

The TP-700 and its predecessors are all cross compatible and capable of using the same exact pads.