



Notice

Veinlite NEO is designed for external examination only

Do not use without a disposable plastic cover

Always change the cover between patients to avoid cross contamination

Do not look directly at the LEDs or shine them directly into anyone's eyes

Do not immerse in liquid

Do not autoclave

Operating Directions

- 1 With LED window facing up slide Veinlite NEO into disposable plastic Cover [F] with flap on bottom.



- 2 Stretch connected flap over the opening.



- 3 Disinfect cover with 70% isopropyl alcohol or medically accepted disinfectant.

- 4 Place LED Window [A] against the skin.

- 5 Depress the End Button [B] to cycle through these modes:

- Orange** Vein
- Green** Artery & Vein
- White** Exam light



- 6 Apply gentle pressure to the Veinlite NEO to ensure it is in contact with the skin.



- 7 There are 3 brightness levels. Depress the Side Button [C] to change the brightness.



- 8 Discard the cover after each use.



- 9 Cap [E] protects the LED Window and End Button while not in use and should be used over the Cover for the best fit. Cap also acts as a lanyard holder.

